# MEDIMAR®

## Eau de mer pure

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#### Isotonic / Hypertonic

#### FORMS AND PRESENTATIONS:

Hypertonic / Isotonic solution based upon seawater presented as a micro-spray vial of 50ml or 150ml usable in any position with a well adapted tip for the nose's anatomy enabling a soft application and a homogenous diffusion until the sinus.

#### COMPOSITION:

- **For an isotonic solution:**  
- Pure seawater equivalent in sodium chloride to ..... 0,9 %
- **For an hypertonic solution:**  
- Pure seawater equivalent in sodium chloride to ..... 2,5 %
- **MEDIMAR®** is naturally rich in sodium (Na<sup>+</sup>), potassium (K<sup>+</sup>), calcium (Ca<sup>2+</sup>), magnesium (Mg<sup>2+</sup>), iron (Fe<sup>2+</sup>), zinc (Zn<sup>2+</sup>), copper (Cu<sup>2+</sup>), chloride (Cl<sup>-</sup>), sulphate (SO<sub>4</sub><sup>2-</sup>), bicarbonate (HCO<sub>3</sub><sup>-</sup>), fluoride (F<sup>-</sup>)...
- Preservative-free.

#### INDICATIONS:

**MEDIMAR® isotonic** is a medical device for baby, child and adult and **MEDIMAR® hypertonic** is a medical device for child and adult. Both are recommended:

- For daily nasal hygiene.
- For pathologies prevention of the ENT area.
- To clear the mucus from blocked and stuffy noses and facilitate wiping.
- To restore the natural humidity of the nose.

**MEDIMAR®** assures an efficient and soft washing for a better breathing thanks to its double mechanical and osmotic action.

#### PRECAUTIONS OF USE:

- Do not inject.
- Do not wash your child's nose while his head is back to avoid coming the liquid down in his throat.
- keep out of children's reach.

#### SIDE EFFECTS

Possibility of transitory feelings of tingling or irritation at the beginning of treatment.

#### POSOLOGY AND METHOD OF ADMINISTRATION

##### Posology:

1 to 2 sprays in each nostril 2 to 6 times daily preferably before each food intake and before sleeping or following the doctor's indications.

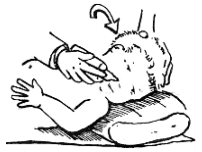
##### Method of administration:

Nasal way.

## MANIPULATION METHODS :

### ➤ For infant and child :

- Turn to the left the child's head who is already lying down.
- Introduce softly the tip into the right nostril.
- Press the tip.
- Turn his head to the other side and repeat the same operation with the second nostril.
- Wipe his nose.
- After use, clean the tip with water and keep the vial and its tip in its packaging.

### ➤ For adult :

- Turn to the right your head.
- Introduce softly the tip into the left nostril.
- Press the tip.
- Repeat the same operation with the second nostril.
- Blow your nose.

- After use, clean the tip with water and keep the vial and its tip in its packaging.

## PARTICULAR STORAGE CONDITIONS:

Store and transport below 30°C temperature.

Medical device, class IIa

Please refer to the explanatory leaflet

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